



PRESCRIPTION MOUNT INSTALLATION INSTRUCTIONS



Please use the following instructions to install your prescription mount after it has been prepared by a licensed optician.

📺 Watch a video of these instructions on the official **YouTube CTRL Channel** here: <https://youtu.be/ZnrB1d5zJH8>

1. REMOVE ORIGINAL NOSE PIECE

Switch the CTRL™ eyewear off (clear state). **NOTE: Hold the eyewear frame while removing the nose piece and avoid pressing the inner liquid crystal surface of the lens.**

Grip the bottom corner of the nose piece and gently rotate outward about 1/4 inch to release from the lens (**Diagram 1**). Repeat process on the opposite side of the nose piece and pull to completely remove (**Diagram 2**).

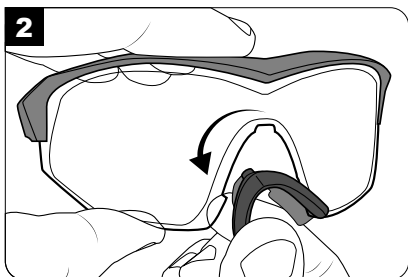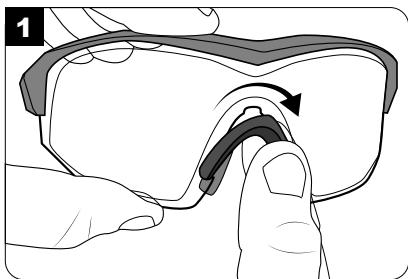

2. INSTALLATION

Slide the nose piece of the prescription mount up into the eyewear lens until it snaps securely into place. **NOTE: Do not remove nose piece from the prescription mount lens.** Follow Step 1 above to remove the prescription mount (by the nose piece) from your CTRL eyewear.

e-TINT LLC

1950 State Route 59, Suite 100 | Kent, OH 44240

www.e-tintproducts.com
